



RECOMMENDED USAGE

Directions

Wipe Out[®] Dairy Wipes should be used as part of your dairy's overall mastitis management program. **Wipe Out**[®] is a tool that will help you maintain optimum pre-milking udder preparation. With **Wipe Out**[®], there's nothing to mix, nothing to measure: you simply pull a pre-soaked towel out of the top of the bucket, wipe the entire teat area, and the cow is ready for milking in seconds.

Wipe Out[®] Dairy Wipes are made from a non-woven material that has been soaked through with a quick-acting and effective antimicrobial solution containing Nisin. Nisin is effective in killing the leading bacteria that cause mastitis, while maintaining excellent teat condition.

Recommended Usage

When used as part of your mastitis prevention program, **Wipe Out**[®] Dairy Wipes provide a true one-step routine that will help you to achieve superior udder disinfection and maximum milk letdown. Prior to milking, all milkers should follow a calm and consistent routine. Udders should be kept clean, as well as having clean cow alleys and clean bedding.

Step 1

Brush loose dirt off the teat.

Step 2

Forestrip onto the cow platform, or into the gutter or strip cup, without getting milk on your hands.

Step 3

Using a new **Wipe Out**[®] Dairy Wipe, and beginning with the teat farthest away from you - wipe down and around the teat to clean the sides of the teat. Repeat on all teats of the cow.

Step 4

Flip over the towel and wipe across the teat opening.

Step 5

Repeat steps 1-4 on additional cows until approximately 60-90 seconds has elapsed from initial contact with the first cow.

Step 6

Attach milking units to the cows (drying is NOT necessary).

Repeat this procedure for all cows, while striving to maintain a lag time of approximately 60-90 seconds between prepping the teats and attaching the milking unit. If procedure is followed properly, the milking units are then placed on clean, dry teats during maximum milk letdown.